

# BD Rowa™ Vmotion Replacing the NUC Mini PC

## Brief Instruction

February 2021  
Version 1.0  
Document ID 1026236171-EN-US



# 1 Replacing the NUC Mini PC




Once the NUC Mini PC has been installed, the Vmotion displays are set up by the BD Rowa Hotline via remote maintenance. You should therefore arrange an appointment with the hotline by telephone prior to the installation.

Scope of delivery:

- NUC Mini PC (Vmotion server)
- Power supply unit


You should proceed as follows to replace a defective NUC Mini PC:

1. Pull the bottom area of the screen forwards.

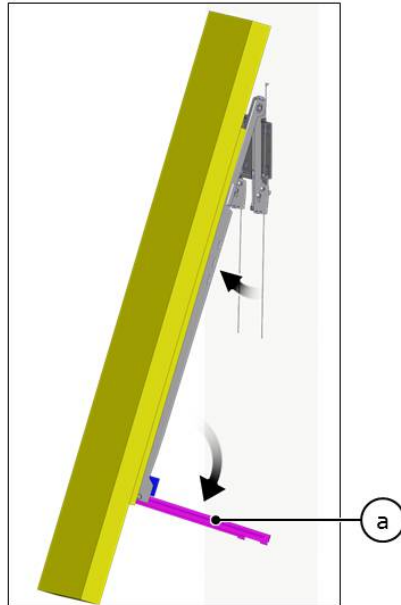

2. Fold out the distance bracket ③.
3. Disconnect the following cables from the defective NUC Mini PC:
  - 1 or 2 HDMI cables (depending on the number of displays connected)
  - 1 or 2 USB cables (depending on the number of displays connected)
  - 1 network cable
  - 1 power cable of the NUC power supply unit
4. Put all the cables that have been disconnected to one side so that they can easily be found after the new NUC Mini PC has been installed.
5. Disconnect the power supply unit of the defective NUC Mini PC from the plug socket and put to one side with the defective NUC Mini PC.
6. Plug the power supply unit of the new NUC Mini PC into the plug socket.
7. Plug the power cable of the NUC power supply unit and all the cables that were previously disconnected into the new NUC Mini PC.
  - Where 2 displays are connected, it doesn't matter which sockets the HDMI cables and USB cables are plugged into.
8. Switch on the NUC Mini PC using the power button.
9. Monitor the start process on the displays.
  - If you cannot see a start process, connect displays with the remote control and wait until Windows has started.

10. Establish the Internet connection.
  - If necessary, have your network passwords ready.
11. Start the TeamViewer application using the **TeamViewer Host** icon.

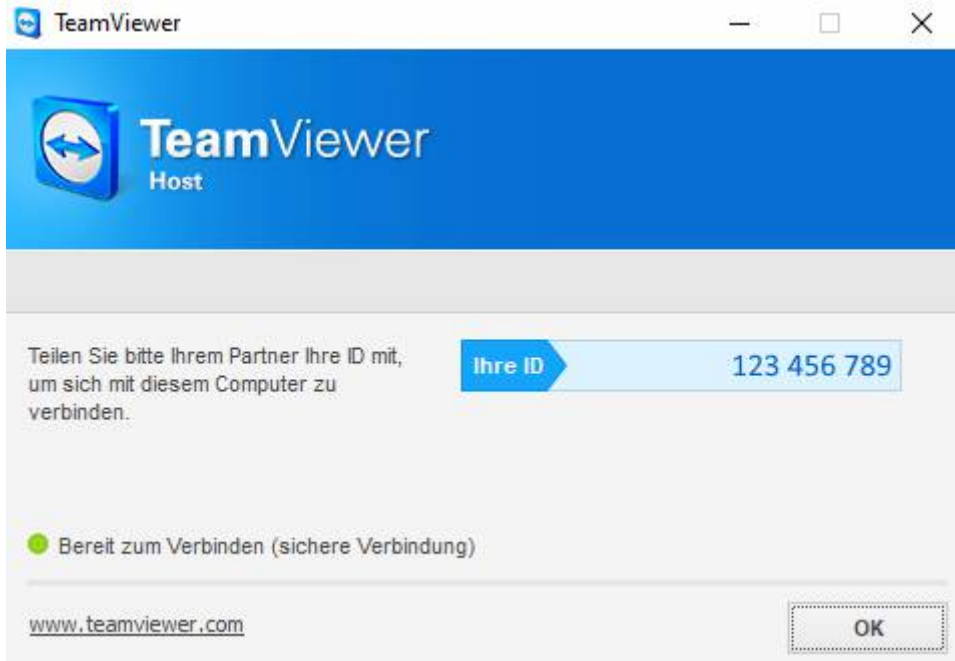

12. Call the BD Rowa Hotline and tell the hotline the sequence of characters under **Your ID**.
13. The rest of the installation is carried out together with the hotline. This process takes approx. 15 minutes.